



PRIME MEDICAL SURGICAL SOLUTIONS

PrimePad Anti-Skid O.R. Pad Series

- For use in any procedure that requires Trendelenburg, Reverse Trendelenburg, or lateral positioning.
- Provides superior support and stability with a next-generation, innovative, high-performance polyurethane foam pad.
- PrimePad material is made with microscopic air capsules that offer immediate rebound, anti-skid properties, enhanced breathability and increased patient support.
- PrimePad material is not temperature sensitive and dissipates heat faster than memory foam.
- PrimePad material provides up to 4 times greater peak pressure reduction compared with memory foam.
- Single patient use due to the dissipation of static properties with each use.
- Single patient use to aid in infection prevention.
- Compatible with patient warming devices.
- Single-packed in compressed format which lowers shipping costs and reduces storage space requirements.
- Available in kits that include: Patient Body Straps, Disposable Draw Sheets, Arm Tucking/Protection Sleeves, and Head Cradles.
- Available in Wing format with integrated arm tucking and protection.

PrimePad-1



Case Pack: 10

PrimePad-1S



Case Pack: 10



Instructions for Use

- First, ensure the O.R. Mattress is secured to the table.
- O.R. Mattress must be clean and dry. Place pad on the table, aligning the pad and mattress perineal cutouts. PrimePads are designed to provide maximum O.R. Mattress coverage.
- Nothing should be placed between O.R. mattress and PrimePad.
- MAXIMUM patient contact with the pad is key to clinical efficacy.
- Ensure the Sacrum, Torso and Shoulders are in DIRECT contact with the pad.
- Minimize lift/draw sheet width to maximize patient contact with the pad.
- Prefer foam or gel donuts over pillows for intra-operative head positioning, as patients' shoulders often rest on pillow instead of the pad. If using pillows, ensure shoulders are in contact with pad.
- Pads with Table Rail Straps Only: The built-in table rail straps should be on the bottom side of the pad. Slide the straps downward between the rail and the table, bring them back up and around the outside of the rail, and affix the strap back upon itself securely. Discard after use.

Not made with natural rubber latex



PrimePad Anti-Skid O.R. Pad Series see full description on pages 7-10

- PrimePad-1:** Anti-Skid Pad w/perineal cutout, 44"x23"x1".
- PrimePad-1S:** Anti-Skid Pad w/perineal cutout w/rail straps, 44"x23"x1".
- PrimePad-2:** 2" Anti-Skid Pad w/perineal cutout, 44"x23"x2".
- PrimePad-2S:** 2" Anti-Skid Pad w/perineal cutout w/rail straps, 44"x23"x2".
- PrimePad-3:** Ramped 1" to 2" Anti-Skid Pad w/perineal cutout, 44"x23"x2".
- PrimePad-3S:** Ramped 1" to 2" Anti-Skid Pad w/perineal cutout, rail straps, 44"x23"x2".
- PrimePad-W1:** Anti-Skid Pad w/integrated 24"x24" arm tucking/protection, perineal cutout, rail straps, 44"x23"x1".
- PrimePad-W2:** Anti-Skid Pad w/integrated 24"x16" arm tucking/protection, perineal cutout, rail straps, 44"x23"x1".
- PrimePad-K1:** Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, 44"x23"x1".
- PrimePad-K2:** Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, 44"x23"x1".
- PrimePad-K3:** Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, arm tucking sleeve pair, 44"x23"x1".
- PrimePad-K4:** Anti-Skid Pad w/perineal cutout, head/shoulder positioner, single body strap set, arm board pair, 44"x23"x1".
- PrimePad-K5:** 2" Anti-Skid Pad w/perineal cutout, head/shoulder positioner, single body strap set, arm tucking sleeve pair, 44"x23"x2".
- PrimePad-K6:** Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, arm tucking sleeve pair, 44"x23"x1".
- PrimePad-K7:** Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, arm tucking sleeve pair, head cradle, 44"x23"x1".
- PrimePad-K8:** Anti-Skid Pad w/integrated 24"x16" arm tucking/protection, perineal cutout, rail straps, single body strap set, disposable draw sheet, head cradle, 44"x23"x1".
- PrimePad-K9:** Anti-Skid Pad w/perineal cutout w/rail straps, disposable draw sheet, 44"x23"x1".
- PrimePad-K10:** 2" Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, 44"x23"x2".
- PrimePad-PED:** Pediatric Anti-Skid Pad, 24"x20"x1".
- PrimePad-SQ1:** Square positioning foam 12"x12"x1".
- PrimePad-SQ2:** 2" Square positioning foam 12"x12"x2".
- PrimePad-BBO:** Bean Bag Overlay, 40"x38"x1".
- PrimePad-CVP:** Cardiovascular Pad, 72"x23"x1".



PRIME MEDICAL SURGICAL SOLUTIONS

PrimePad Anti-Skid O.R. Pad with Integrated Arm Tucking and Protection

- Provides all of the features and benefits of our PrimePad Series, with the added feature of integrated arm tucking and protection Wings.
- The integrated arm tucking and protection offers patient safety and clinical staff ease of use and convenience in patient positioning for lithotomy/ steep Trendelenburg.
- The Wing material can be used to either fully encapsulate the patients' arms, or they can be used to simply pad the patients' arms, depending upon hospital protocol.
- Any excess or unnecessary Wing material can be removed by simply tearing it, and it easily tears in a linear plane.
- Allows the Anesthesia provider easy access to the IV site, and Wing material allows for channeling of IV tubing and pulse oximetry cables by tearing a window or slit in the Wing material.
- Eliminates the need to use a draw sheet for arm tucking and allows for 100% patient contact with the pad.
- 2 options with different widths in arm tucking/ protection material:

PrimePad-W1 Wing dimensions:
24" long x 24" wide

PrimePad-W2 Wing dimensions:
24" long x 16" wide

PrimePad-W2



Case Pack: 4



Providing a Better Way

The PrimePad is a new innovation in anti-skid patient positioning, offering a next-generation, innovative, high-performance polyurethane foam pad. PrimePad offers excellent anti-skid properties, pressure reduction and patient support, providing confidence to the surgeon and staff that the patient will remain immobile.

Instructions for Use

Follow the Instructions For Use for the PrimePad Series of Anti-Skid Pads for installation of the pad on the O.R. Mattress.

Once the patient is properly positioned on the PrimePad, a draw sheet is not required to tuck and protect the arms. To allow for 100% patient-to-pad contact, remove the draw sheet and utilize the Wing material to tuck/protect the arms.

To utilize the Wing material for arm tucking and protection, bring the full length of the pad to the patient's side to determine how much length is needed. If excess Wing material needs to be removed, simply tear off the un-needed material. The material tears in linear plane, and tearing reduces the need for scissors.



One method for utilizing the Wing material is to have a team member on each side of patient. One team member will slightly "log roll" the patient toward them, while the other team member wraps the patient's arm in the Wing and then places the end of the Wing under the patient's torso. Repeat the same steps on the opposite side.

The PrimePad material's flexibility and high coefficient of static friction provides secure arm tucking and protection without the need for external accessories such as Velcro, straps and draw/lift sheets.

Not made with natural rubber latex



For PrimePads with Integrated Arm Tucking and Protection:

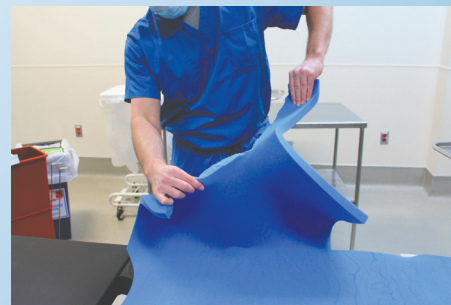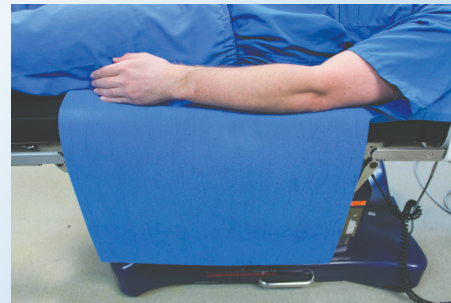

PrimePad Series

All PrimePads are folded, compressed and vacuum sealed for compact storage and shipping efficiency.



PrimePad-1
Case Pack: 10



Anti-Skid Pad w/perineal cutout, 44"x23"x1"

PrimePad-1S
Case Pack: 10



Anti-Skid Pad w/perineal cutout w/rail straps, 44"x23"x1"

PrimePad-2
Case Pack: 4



2" Anti-Skid Pad w/perineal cutout, 44"x23"x2"

PrimePad-2S
Case Pack: 4



2" Anti-Skid Pad w/perineal cutout w/rail straps, 44"x23"x2"

PrimePad-3
Case Pack: 4



Ramped 1" to 2" Anti-Skid Pad w/perineal cutout, 44"x23"x2"

PrimePad-3S
Case Pack: 4



Ramped 1" to 2" Anti-Skid Pad w/perineal cutout, rail straps, 44"x23"x2"

PrimePad-W1

Case Pack: 4



Anti-Skid Pad w/integrated 24"x24" arm tucking/protection, perineal cutout, rail straps, 44"x23"x1"

PrimePad-W2

Case Pack: 4



Anti-Skid Pad w/integrated 24"x16" arm tucking/protection, perineal cutout, rail straps, 44"x23"x1"

PrimePad-K1


Case Pack: 10



Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, 44"x23"x1"

PrimePad-K2


Case Pack: 10

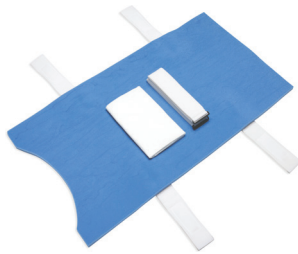

Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, 44"x23"x1"

PrimePad-K3

Case Pack: 3

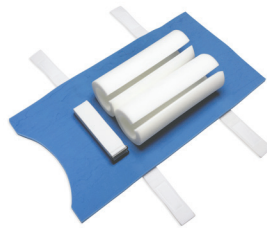

Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, arm tucking sleeve pair, 44"x23"x1"

PrimePad-K4

Case Pack: 3



Anti-Skid Pad w/perineal cutout, head/shoulder positioner, single body strap set, arm board pair, 44"x23"x1"

PrimePad-K5


Case Pack: 3



2" Anti-Skid Pad w/perineal cutout, head/shoulder positioner, set of two body straps, arm tucking sleeve pair, 44"x23"x2"

PrimePad-K6


Case Pack: 4

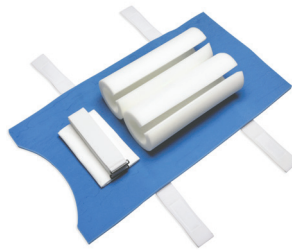

Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, arm tucking sleeve pair, 44"x23"x1"

PrimePad-K7

Case Pack: 3

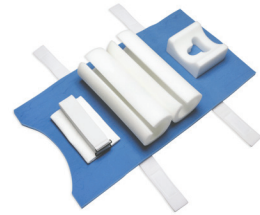

Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, arm tucking sleeve pair, head cradle, 44"x23"x1"

PrimePad-K8

Case Pack: 3



Anti-Skid Pad w/integrated 24"x16" arm tucking/protection, perineal cutout, rail straps, single body strap set, disposable draw sheet, head cradle, 44"x23"x1"

PrimePad-K9


Case Pack: 10



Anti-Skid Pad w/perineal cutout w/rail straps, disposable draw sheet, 44"x23"x1"

PrimePad-K10

Case Pack: 4

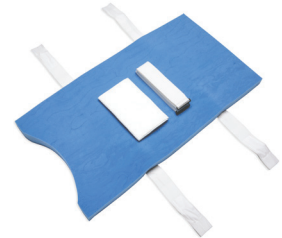

2" Anti-Skid Pad w/perineal cutout w/rail straps, single body strap set, disposable draw sheet, 44"x23"x2"

PrimePad-PED

Case Pack: 10



Pediatric Anti-Skid Pad,
24"x20"x1"

PrimePad-SQ1

Case Pack: 25



Square Positioning Foam
12"x12"x1"

PrimePad-SQ2

Case Pack: 25



2" Square Positioning
Foam 12"x12"x2"

PrimePad-BBO

Case Pack: 6



Bean Bag Overlay,
40"x38"x1"

PrimePad-CVP

Case Pack: 4



Cardiovascular Pad,
72"x23"x1"