



### **PrimePad® Series Instructions for Use:**

- First, ensure the O.R. Mattress is secured to the table.
- O.R. Mattress must be clean and dry. Place pad on the table, aligning the pad and mattress perineal cutouts. PrimePads® are designed to provide maximum O.R. Mattress coverage.
- Nothing should be placed between O.R. mattress and PrimePad®.
- MAXIMUM patient contact with the pad is key to clinical efficacy.
- Ensure the Sacrum, Torso and Shoulders are in DIRECT contact with the pad.
- Minimize lift/draw sheet width to maximize patient contact with the pad.
- Prefer foam or gel donuts over pillows for intraoperative head positioning, as patients' shoulders often rest on pillow instead of the pad. If using pillows, ensure shoulders are in contact with pad.
- Pads with Table Rail Straps Only: The built-in table rail straps should be on the bottom side of the pad. Slide the straps downward between the rail and the table, bring them back up and around the outside of the rail, and affix the strap back upon itself securely. Discard after use.

Not made with natural rubber latex

