



## **PrimePad® Series – Wing Pad Instructions For Use:**

Follow the Instructions For Use for the PrimePad® Series for installation of the pad on the O.R. Mattress.

Once the patient is properly positioned on the PrimePad®, a draw sheet is not required to tuck and protect the arms. To allow for 100% patient-to-pad contact, remove the draw sheet and utilize the Wing material to tuck/protect the arms.

To utilize the Wing material for arm tucking and protection, bring the full length of the pad to the patient's side to determine how much length is needed. If excess Wing material needs to be removed, simply tear off the un-needed material. The material tears in linear plane, and tearing reduces the need for scissors.

One method for utilizing the Wing material is to have a team member on each side of patient. One team member will slightly "log roll" the patient toward them, while the other team member wraps the patient's arm in the Wing and then places the end of the Wing under the patient's torso. Repeat the same steps on the opposite side.

The PrimePad® material's flexibility and high coefficient of static friction provides secure arm tucking and protection without the need for external accessories such as Velcro, straps and draw/lift sheet.

Not made with natural rubber latex

